# Nasal Aspirator

For baby's Nasal Secretion

Caution

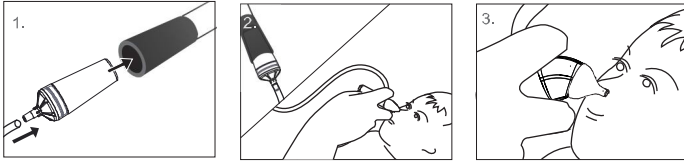
The product only can be operated by adults. Potential choking hazard, keep all parts out of reach of children.

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
## Vacuum use (EN)

1. Assemble the device and insert the vacuum connector into the tube of the vacuum cleaner.
2. Placing baby on his/her back in a stable and comfortable position. Gently place the soft tip to baby's nostril.
3. Switch on the vacuum cleaner and adjust the necessary sucking force with the vacuum modulator of the vacuum cleaner.

## Mouth use

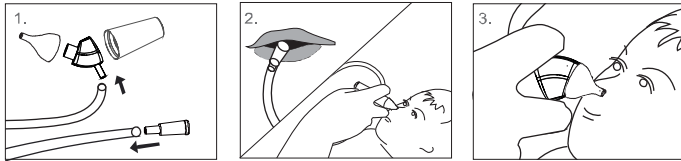
1. Connect mouth piece and hose, placing baby on his/her back in a stable and comfortable position.
2. Place mouth piece in mouth while holding baby with one hand and gently place the soft tip to baby's nostril, forming a seal.
3. Always begin by sucking gently and slowly until all loose mucus has been removed.

## How to clean after use

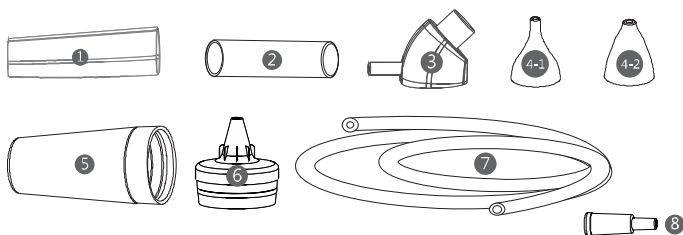Disassemble Baby Aspirator, flush each part under running water after use. (All parts, except the external tank and cap, can be sterilized in boiled water.) After drying, keep the parts fitted and in a clean place.

| Part No.    | Material | sterilized boiled by water |
|-------------|----------|----------------------------|
| 5 6 8       | PP       | O                          |
| 1 3         | PS       | X                          |
| 2 4-1 4-2 7 | Silicone | O                          |

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