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EN instructions for use

Store the casing and the user instructions for future reference.

REUSABLE CUSHION FOR HOT/COLD THERAPY
ThermoGel taps the natural healing power of heat and cold in an easy and comfortable way. ThermoGel is reusable and can serve as a valid substitute for an ice pack and/or a hot water bottle.
Medical Equipment.

WHEN TO USE IT
Cold Therapy
For minor bruises and acute injuries (pulled muscles, muscular tears, sprains), headache or toothache, localised skin inflammation. It is also indicated for insect bites.

Heat therapy
It is indicated for muscle pain (e.g., contracture, cramps) and joint pain (e.g., rigid joints).

THE THERMOGEL RANGE
REF 58012458000000 - THERMOGEL cm 10x10
reusable cushion for hot/cold therapy with **comfortable material** that is pleasing to the touch. Can be applied directly to the area to be treated.
REF 58012455000000 - THERMOGEL cm 10x26
reusable cushion for hot/cold therapy with **comfortable material** that is pleasing to the touch, with a robust, extendible elastic strap. Thanks to the elastic band it can be applied directly to the area to be treated and will remain firmly in place.
REF 58012454000000 - THERMOGEL cm 10x26
reusable cushion for hot/cold therapy **with protective cover** in non-woven fabric, for protecting the skin from the direct application of heat and cold.
REF 58012456000000 - THERMOGEL cm 20x30
reusable cushion for hot/cold therapy **with cover** made of polar fleece and **elastic bands** to easily anchor the cushion to the area to be treated. The cover helps protect the skin from the direct application of heat or cold.
REF 58012457000000 - THERMOGEL cm 17x30
reusable cushion for hot/cold therapy with **comfortable material** and an **ergonomic shape** for the **knee** that adapts perfectly to this joint, which is often difficult to treat. The practical adjustable **elastic band** helps keep the cushion in the proper position guaranteeing freedom of movement. It can be applied directly to the area to be treated.

HOW IT'S USED
For **cold therapy** place ThermoGel in the freezer for at least two hours.
For **heat therapy** see the table below:

REF	DESCRIPTION	IMMERSION TIME IN BOILING WATER	500W MICRO-WAVE**	700W MICRO-WAVE**
58012458000000	THERMOGEL 10 X 10 CM BLUE FABRIC	45"	20"	15"
58012455000000	THERMOGEL 10 X 26 CM BLUE FABRIC	60"	30"	25"
58012457000000	THERMOGEL 17X30 CM BLUE FABRIC	45"	45"	40"
58012454000000	THERMOGEL 10 X 26 CM TRANSPARENT	30"	1'10"	40"
58012456000000	THERMOGEL 20 X 30 CM TRANSPARENT	20"	1'40"	1'30"

* After immersion, dry the reusable cushion before applying to the affected area.
** It is not necessary to immerse the product in water when heating in a microwave.
Check the condition of the device before use.

WARNING
• ThermoGel must NOT:
- be used by children without the supervision of a doctor or of an adult
- be used by people who are not in full possession of their mental/physical faculties must be assisted when using the device
- be used by people with reduced sensitivity to heat/cold
- be used on newborns
- be considered a toy
- be used together with creams, ointments or similar products
- be ingested
- be used after it expired (see expiry date indicated on the product and/or casing).
• Keep out of reach of children.
• Do not damage the casing and ensure the gel does not come out: If broken, dispose of them as normal household waste.
• If the contents comes into contact with the eyes or skin, rinse thoroughly with water and consult a doctor.
• Consult a doctor before each use and in the event of prolonged use.
• Consult a doctor if symptoms persist.
• If this product is used for a purpose other than that intended, or is not used in compliance with the instructions for use, the user will be considered solely responsible for any consequences.
• As such, any damage to the product or other consequences are not covered by the right to lodge a complaint.
• Store at room temperature away from sources of heat and away from sunlight.
• Report any serious accident occurring in connection with the device to your competent authority and to the manufacturer.

DISPOSAL
The product must not be dispersed into groundwater, watercourses or sewers.
Observe applicable local laws for disposal.

PRECAUTIONS FOR USE
Read usage instructions carefully.
Each individual application on the affected area must not last for more than 20 minutes; before applying to the affected area again wait at least 60 minutes.
For individuals who have particularly sensitive skin, wrap the ThermoGel bag in a cloth to avoid injuries resulting from heat or cold.
Do not apply on wounds.
Apply the elastic band (where present) with only minimal tension.
After each use wash the cover (where present) with a gentle detergent and allow it to dry completely.

The gel does not freeze: even if it is stored in the freezer, ThermoGel remains soft and shapeable.

Hand wash in cold water.

CE	Symbol of compliance with REGULATION (EU) 2017/745	!	Warning!
REF	Product ID Code	Der Grune Punkt	
LOT	Production Batch	Use by date - Expiry date	
MD	Medical Equipment	Please refer to the instructions for use	
Fab	Legal Manufacturer	Recyclable packaging	
21 PAP	Material identification Packaging components	UDI	Unique identifier of the equipment
EC REP	European Authorized Representative	Importer	
Sun	Keep away from sunlight	Keep dry	
No	Do not use if packaging is not intact and read the instructions for use		

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