GEL Oraflologo®

(SL) ORAFLOGO® gel

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(EN) ORAFLOGO® gel

This leaflet provides some useful informations. Please read it carefully before you start using ORAFLOGO® Gel and respect the instructions for use. If you have any further questions, ask your doctor, dentist or pharmacist.

DESCRIPTION

Aphthae or mouth lesions can cause pain and discomfort.

ORAFLOGO® Gel relieves the pain, forms a protective layer and seals oral injured tissues, allowing natural healing.

ORAFLOGO® Gel acts through a physical process and it's particularly useful to a controlled local treatment.

ORAFLOGO® Gel can be easily and directly applied in the lesions.

By coating the top of the ulcer or mouth lesion, it forms an unnoticeable shield over damaged tissues. ORAFLOGO® Gel can reduce the irritation caused by consuming solid food and fluids.

In addition to forming a physical barrier over sensitive tissues ORAFLOGO® provides lubrication and hydration to the oral mucosa.

Treatment with ORAFLOGO® Gel should be initiated as soon as signs and symptoms of oral lesions appear.

ORAFLOGO® Gel is suitable for use by adults and children over 2 years of age.

INDICATIONS

ORAFLOGO® Gel is used for the treatment and management of oral conditions of various etiologies: gingivitis, stomatitis, gingival recession and all types of oral wounds (mouth sores, aphthous ulcers and traumatic ulcers), including those caused by oral surgery or ill-fitting dentures or braces.

INGREDIENTS

Purified Water, Macadamia ternifolia seed oil, Aesculus hippocastanum, Tocopheryl acetate, Centella asiatica extract, Triethanolamine, Carbopol, Trigonella foenum-graecum extract, Aloe Barbadensis gel, Imidazolidinyl urea, Sodium Hyaluronate, Phenoxylethanol, Sodium dehydroacetate, Allantoin, Cupressus sempervirens oil, Melaleuca alternifolia oil, Thymus Zygis oil, Ubiquinone, Propylene glycol, Ascorbic acid, Panthenol, Ethylparaben, Methylparaben, Propylparaben.

INSTRUCTIONS FOR USE

ORAFLOGO® Gel can be used 3 times a day after performing oral care (e.g. brushing teeth), unless otherwise prescribed.

Wash your hands before and after application.

Squeeze a small amount of gel onto a clean, dry fingertip or cotton bud, sufficient to cover the affected area, and then apply it directly to the lesion, gently.

Continue applications up to complete healing.

ORAFLOGO® Gel is hygroscopic i.e. it seeks out water. If you dry the affected area before application with the help of a cotton swab, you are soaking the excess of water (e.g. saliva) on the surface, allowing ORAFLOGO® Gel to act quickly.

On occasions, eating or drinking may make it necessary to re-apply the gel.

PRECAUTIONS:

ORAFLOGO® Gel is for local use on the mucosa in the mouth only. Do not swallow.

Do not use ORAFLOGO® Gel if you are allergic (hypersensitive) to any of the ingredients listed.

If giving ORAFLOGO® Gel to a child, you should not place the gel in the back of the tongue or throat area.

ORAFLOGO® Gel is not known to affect, or to be affected by, any other medicines.

Avoid contact with eyes. In the event of accidental contact, rinse thoroughly with plenty of water.

You must contact a doctor, dentist or pharmacist if your symptoms worsen or do not improve.

If you are pregnant or breastfeeding, talk to your doctor, dentist or pharmacist before using ORAFLOGO® Gel.

STORAGE CONDITIONS

Store at room temperature in a dry place. Maximum 25°C. Do not refrigerate.

Do not use the product after the expiry date shown on the packaging. Keep out of reach and sight of children.

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Danmark - Medic Danmark A/S, Korskindelund 4, DK-2670 Greve, Slovenia - Panakea d.o.o., Savska cesta 10, 1000 Ljubljana